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English
Wear’N’Go
Calf Compression Sleeves
Function:
 Provides firm and even compression to the calf muscles, helping to relieve discomfort, improve blood circulation, support muscle recovery, and reduce the risk of future injuries. Suitable for both use during sports activities and for post-exercise recovery.

Instructions for Use:

- Grasp the product and gently pull it over the foot and up the calf.
- Adjust the sleeve so that it fits evenly and covers the area from the top of the ankle to below the knee. Ensure the material lies smoothly without folds.
- To remove, grip the top – just below the knee – and carefully pull it downward and off the foot.

Sizes:

- Measure as described on the packaging.
- Find the correct size using the size guide on the packaging.

Materials:

See the packaging.

Washing Instructions:

Follow the instructions on the packaging.

Important Information:

- Use with care and appropriately.
- Do not modify or impair the product’s function.

- Not to be used by individuals sensitive to any component of the product.
- The product is not flame-resistant.
- Avoid temperatures above 120°C / 248°F.
- Use only for the purpose described above.
- Recommended usage time is a maximum of 3–4 hours at a time.
- Try the product to ensure proper fit.
- Contact the manufacturer or your local supplier for any questions.
- All users are advised to consult a healthcare professional before use. Athletes should consult a coach, physiotherapist, or physician.
- Discontinue use and consult healthcare personnel if pain, skin irritation, injuries, or unusual reactions occur.
- Do not use directly on open wounds.
- Intended for individuals over 18 years of age.
- Use at your own risk.

Disposal:

Dispose of the product and packaging in accordance with local or national environmental regulations.

Liability:

The manufacturer disclaims liability in the following cases:

- The product is not maintained as described in the instructions.
- The product is combined with components from other manufacturers.
- The product is used outside recommended conditions, applications, or environments.
- Damage occurs due to incorrect use.

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