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English Wear’N’Go Elbow Support Function:

Provides light support, compression, warmth, and stability to the elbow. The material increases blood circulation and relieves discomfort. The product is well-suited for use in both sports and rehabilitation after injuries or ailments.

Instructions for Use:

- Hold the product and pull it up to the elbow.
- Position the product so that the silicone pads are placed above and below the elbow. The silicone pads should not cover the tip of the elbow (olecranon).
- To remove the product, grip the bottom and pull it off the elbow/arm.

Sizes:

- Measure as described on the packaging.
- Determine the correct size using the size guide on the packaging.

Materials:

See the packaging.

Washing Instructions: Follow the instructions on the packaging.

Important Information:

- Use the product carefully and appropriately.
- Do not modify the product inappropriately / The function must not be altered or weakened.
- Do not use if you are hypersensitive to any of the product’s components.
- The product is not flame-resistant.
- Avoid temperatures above 120°C / 248°F.
- Use only as described.

- Recommended usage time is max. 3–4 hours.
- Ensure the product fits properly.
- Contact manufacturer or local supplier if you have questions.
- Consult healthcare professionals before use. Athletes should consult a coach, physiotherapist, or doctor.
- Discontinue use and consult healthcare professionals if pain, skin irritation, injury, or unusual reactions occur.
- Do not use on open wounds.
- For use by individuals over 18 years.
- Use at your own risk.

Dispose:

Dispose of the product and packaging according to local or national environmental regulations.

Liability Disclaimer:

The manufacturer assumes no responsibility for:

- Improper maintenance.
- Use with components from other manufactur-ers.
- Use outside recommended conditions, applications, or environments.
- Injuries caused by incorrect use.

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